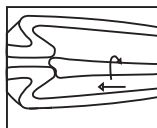
CellPreserv

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
1. Using the plastic spatula, you will obtain samples from the ectocervix and from the bottom of the vaginal sac.

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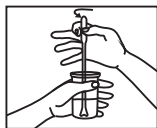
2. Immediately insert the spatula into the vial containing the CellPreserv Solution and shake vigorously to release the collected cells. After this, discard the spatula.

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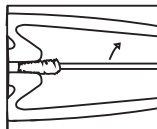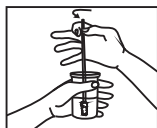
3. With the cervical brush, you will obtain samples from the endocervix. Insert the brush into the endocervical canal until only the last bristles are visible, slowly rotate 1/2 turn in one direction only.

Warning: Do not rotate the brush more than one full turn as this may cause bleeding. Do not use on pregnant patients.

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4. Shake the brush vigorously inside the CellPreserv solution and rub the bristles on the vial wall to help release the collected cells. After this, discard the brush.

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5. Close bottle tightly to prevent leakage.

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6. Fill in the patient data as indicated.

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IMPORTANT INFORMATION

Composition

The CellPreserv solution is a methanol-based solution designed to conserve the collected cytology material to liquid base.

Storage

Before Collection:
Temperature: 15° to 30°C
Validity: 2 years

After collection:
Temperature: 4° to 37°C
Validity: 6 weeks

Transport

Make sure that the bottle containing the collected material is tightly closed.

Precautions

The CellPreserv solution contains methanol, a harmful substance which, if swallowed, may cause blindness or be fatal. Keep away from fire, heat and sparks.

Warnings

Do not use lubricants when making the collection.
Prior to the collection, remove excess mucus, inflammatory exudate or other discharge.
The uterine lap should not be cleaned with saline wash.
If you want to perform colposcopy, collect before applying acetic acid or iodine.

The presence of abnormal blood or any vaginal content may interfere with CellPreserv results.

The patient should not use vaginal medication, vaginal contraceptives or take intimate showers within 48 hours prior to the collection.

Sexual abstinence is required within 72 hours prior to the collection.

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Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatara, BKR 4013 Malta